



DERM Plus range

CryoIQ's most popular devices for medical professional day to day use. Uses cartridges with valves. Freezing tip fixed, not exchangeable.



DERM Plus model

CryoIQ® | DERM Plus for spray freezing
CryoIQ® | DERM Plus for contact freezing ø1, 3, 5 mm



PRO range

For the specialists that works with more advanced cryo-surgery. Large selection of exchangeable freezing tips for different type of skin lesions. Uses cartridges with valves.



PRO model

CryoIQ® | PRO for spray freezing
CryoIQ® | PRO for contact freezing ø1, 3, 5, 7 mm



Gas Cartridges with OxiBlock® Technologies

The new coating OxiBlock® on the inside of the cartridge gives you a contamination-free environment for the gas. With this technique, we are the first brand that can offer you a cartridge that meets the new upcoming Medical Device Directive.

What we have seen is that without our coating you are not able to obtain a contamination free surface inside. This effects the durability and purity of the gas.



Address

CryoIQ global
Apelrödsvägen 1
439 32 Onsala
Sweden

Telephone

+46-31 400 500

Fax

+46 300-56 94 99

E-mail

info@cryoiq.com

Website

www.cryoiq.com

CryoIQ global is the producer of the CryoIQ technology. All products are Made In Sweden.

The products are design protected all over the world.

CryoIQ® and OxiBlock® are registered trademarks of CryoIQ global

*For more information about our products
visit us online at www.cryoiq.com
or talk with your local
CryoIQ® representative*

Representative



CryoSurgery
The Innovative Way




Innovative cryosurgery at -89°C

CryolQ® is a portable and effective device for removal of warts, skin tags, fibroma, keratosis, hemangioma, solar lentigo and other common skin lesions. Cryosurgery treatment is recognized as a low pain treatment without need for preparation or follow-up care. The CryolQ® devices have been developed during a long time and thus have a unique performance. The unique cartridge construction with the built-in valve prevents leakage of content. CryolQ® devices are safe to use and do not require any maintenance at all.

Spray Freezing®

CryolQ® uses Spray Freezing® where N₂O (nitrous oxide) is brought directly through a micro capillary to the treatment site. The freezing effect and temperature are constant at -89°C during the treatment. Freezing of a tissue to a depth of 3 mm by a diameter of 10 mm will be reached after approximately 12 seconds.

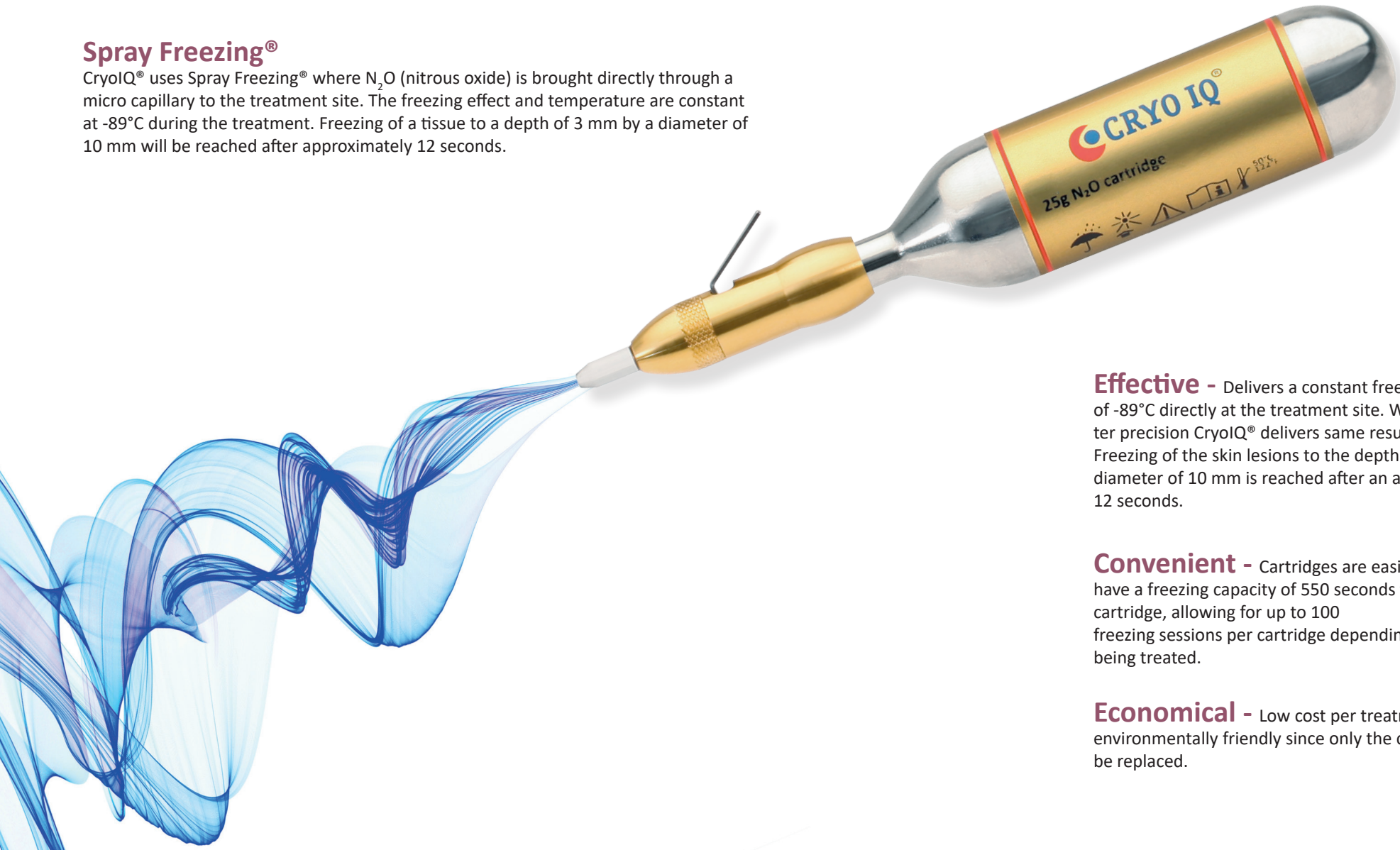

Effective - Delivers a constant freezing temperature of -89°C directly at the treatment site. With a millimeter precision CryolQ® delivers same result every time. Freezing of the skin lesions to the depth of 3 mm by a diameter of 10 mm is reached after an application time of 12 seconds.

Convenient - Cartridges are easily exchanged and have a freezing capacity of 550 seconds for the 25 gram cartridge, allowing for up to 100 freezing sessions per cartridge depending on the lesion being treated.


Economical - Low cost per treatment, environmentally friendly since only the cartridge has to be replaced.

Selection of accessories

Wide range of freezing tips developed for various applications designed for CryolQ® | PRO


Tip for Spray Freezing

Glass tip 60µ, 80µ, 100µ capillary length 22 mm

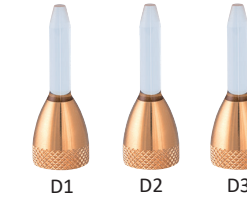

Gynecology tip for Spray Freezing

Glass tip wide ø4 mm, length 13 cm, straight or angled 5°

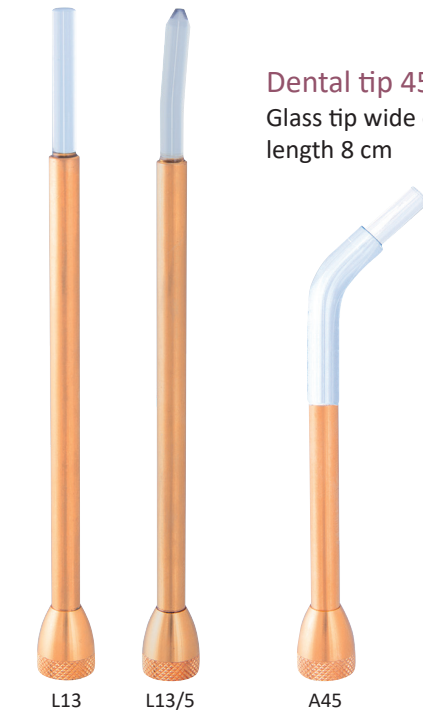


Dental tip 45°

Glass tip wide ø4 mm, length 8 cm

Tip for contact freezing

Contact surface ø1, 3, 5, 7 mm

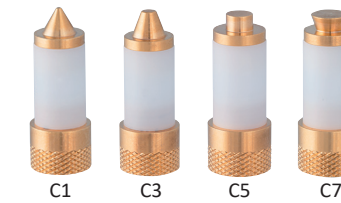

LED Light

Available for models with 25 gram cartridges



Gas Cartridges 25 gram

CryolQ® offers cartridges, with valve. The N₂O gas is of medical quality, filtered to 5 µm. Equipped with Oxiblock® technology.



Packed in box of 1 or 4 cartridges



VETERINARY



PODIATRY



DERMATOLOGY



DENTISTRY

